

STEAM INDICATOR (Autoclave) TAPE

For Monitoring Steam Sterilisation Processes

Excelsior Code: TAP-SCS-E



Product Description

Excelsior Steam autoclave indicator tape contains no lead or other toxic heavy metals and are latex free. The autoclave tape is manufactured for closing packages, instrument wraps, and other items requiring steam sterilisation. The autoclave tape can be utilised for identifying processed and non processed items up to 140°C. The tape adhesive can adhere to paper, plastic, glass, metal, and non-woven materials.

Physical Properties

Process	Steam
Tape Dimensions	18mm x 50m (0.75" x 55 yards)
Packaging	12 Rolls
Chemical Indicator	Initial Colour: White Signal Colour: Dark / Black Chemical Indicator Inks contain no lead or other toxic heavy metals

Indications for Use

Type 1 Indicator :

- 121°C, ≥ 10 minutes
- 134°C, ≥ 2 minutes

FDA cleared cycles:

- 121°C, ≥ 30 minutes (gravity)
- 132°C, 4 minutes (pre-vacuum)
- 135°C, 3 minutes (pre-vacuum)

Instructions for Use

Use a minimum of a three (3) inch section of tape to close packages/items intended for steam sterilisation. Process packages/items as instructed in the autoclave/steriliser validation or manual.

Upon exposure to high temperature steam, the chemical indicator will transition from white to dark. The transition colour may vary depending on the load configuration, length and conditions of exposure. A colour transition from white to dark provides indication of exposure to high temperature steam.

The chemical reaction which causes the colour transition is a steam specific reaction and is irreversible.

Performance Characteristics

Result Availability	Immediately following exposure to high temperature steam processes
---------------------	--



***Colours shown are representations of printed ink initial and signal colours but may vary from actual use.**

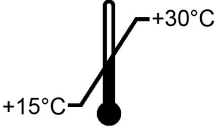







The signal colour achieved from exposure to steam may vary from the example above due to differences in processing parameters (i.e. load content, steam quality, cycle time, temperature, etc.).

Compliance

ISO 11140-1:2014 Sterilization of health care products – Chemical Indicators- Part 1: General Requirements for Type 1 Chemical Process Indicators

Shelf Life and Storage

	15°C to 30°C		Keep away from sunlight
	20% to 80% relative humidity		Keep Dry
Shelf Life	2 years from the date of manufacture. The date of manufacture is based on the day the indicating ink is applied to the substrate. The remaining shelf life upon receipt will be shorter than 2 years		Do not reuse
	Do not use the product if the indicator has transitioned from white to dark. Keep away from sterilants. Do not use after expiration date.		

Disposal

Discard as general waste.